

For every need  
there is a **SenSura**



Coloplast A/S  
Holtedam 1  
3050 Humlebæk  
Denmark

[www.coloplast.com](http://www.coloplast.com)

Coloplast, SenSura and Hide-away are registered trade marks of Coloplast A/S. VELCRO is a registered trade mark of Velcro Industries B.V. © 2007/12. All rights reserved Coloplast A/S, DK- 3050 Humlebæk, Denmark. Manufacturer: Coloplast A/S, DK-3050 Humlebæk, Denmark.

**SenSura**  
Designed to meet  
your needs



**SenSura**



# SenSura was developed in close cooperation with users and nurses


Since Coloplast introduced the first disposable ostomy bag in 1957 we like to think that we know more about the challenges of living with a stoma than most in this field. However, we will never know as much as the people that are using our solutions. That is why listening and responding to user needs has always been part of our way of doing business.

Thanks to this principle the real credit for many of Coloplast's most important contributions to world class ostomy care actually belongs to people living with a stoma and their care givers. That's how **SenSura**, the new standard in stoma care, was developed.

## SenSura has solved the adhesive dilemma once and for all

When we set out to develop the **SenSura** concept our aim was to offer a significantly better solution to people with a stoma in terms of security. Security in the sense that the appliance will not leak and that it will not damage the skin. And to achieve this we had to solve "the adhesive dilemma" that people with stomas have had to live with the last 50 years.


The solution to the dilemma is the revolutionary double layer adhesive that offers double security in two ways. It keeps the appliance tightly fixed to the skin AND allows for easy and painless removal. And it prevents stoma output from leaking unto your skin at the same time as it absorbs sweat and other fluids from eroding the adhesive effect.

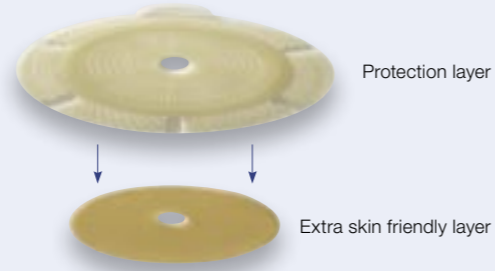

The double-layer adhesive of the **SenSura** base plate keeps the stoma appliance safely in place and protects the skin at the same time.

## SenSura provides confidence for every individual need

As presented on the following pages the **SenSura** range covers solutions for any individual needs – even the special ones. All solutions are based on the unique double layer **SenSura** adhesive that fulfils all the most important demands expressed by users and care givers all over the world. We have improved five important functions of the adhesive for your security. Together these improvements form the Circle of Security that supports your confidence.

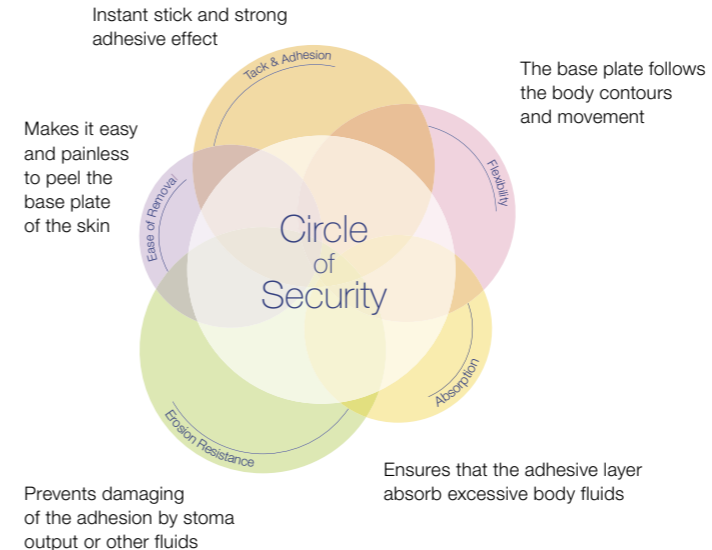

“If your stoma device can’t make you feel confident, nothing else matters”



# Everybody has the right to a stoma care solution that fits their individual needs

The **SenSura** range offers a wide range of stoma appliances. 1-piece solutions where bag and adhesive plate are integrated and 2-piece solutions where the adhesive base plate stays on the skin for several days and only the bag is changed when necessary.

**SenSura** offers you the choice of 2 types of 2-piece appliances: The **SenSura Flex** 2-piece with adhesive coupling and **SenSura Click** with mechanical coupling. The range also covers a number of different versions and sizes so that you have more than 100 **SenSura** solutions to choose from. These solutions can be combined with the oval Convex light shell for special needs. This allows you to choose exactly the solution that suits your individual needs whatever your preferences are.

“I am not a standard person. So I won’t settle for a standard solution”



### SenSura 1-piece system

Skin protection, skin friendliness and convenience in one. Comes with open or closed bags and may be combined with the special oval Convex light base plate.



### SenSura Flex 2-piece system

The adhesive coupling system that provides the ultimate combination of security and flexibility. Comes with open or closed bags and may be combined with the special oval Convex light base plate.



### SenSura Click 2-piece system

The mechanical coupling system that provides extra reassurance with audible confirmation of coupling lock. Comes with open or closed bags and may be combined with the special oval Convex light base plate.



### SenSura Convex light

The oval shape of the Convex light base plate helps overcoming the challenges of a retracted stoma by providing extra support in the peristomal area and can be combined with all **SenSura** solutions. Ask your nurse for further information about **SenSura** Convex light.



## SenSura 1-piece system

Your choice for skin protection, skin friendliness and convenience in one


1-piece appliances are mostly chosen because of convenience. Some find it easier to change the whole appliance every time, and 1-piece appliances are usually more discrete than 2-piece solutions. Therefore, ease of use and flat design are important properties. And since 1-piece appliances are changed frequently, it is vital that changes are comfortable and painless and that the adhesion works instantly.

### The optimal 1-piece solution thanks to the double layer adhesive

The unique properties of the **SenSura** double layer adhesive ensures that the appliance sticks instantly while protecting your skin from stoma output. At the same time the special removal ear makes it easy to peel the adhesive off without causing pain and without leaving residues on the skin. Actually a study comparing **SenSura** 1-piece with another modern appliance showed that majority of the users found that **SenSura** was less painful to remove.<sup>1</sup>



The special removal ear of the **SenSura** 1-piece appliance.

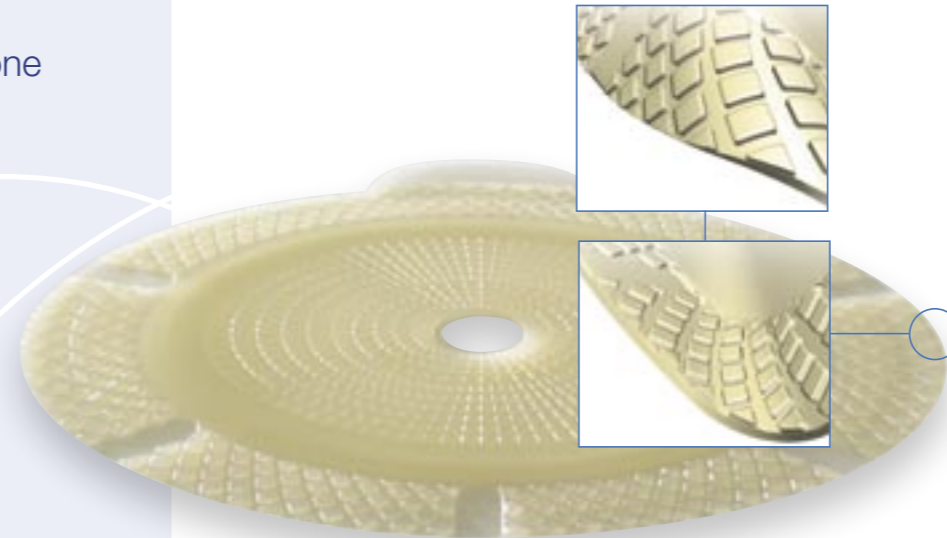

### Flexible, flat and discrete

The special flat Flex pattern on the base plate makes the appliance very flexible and conformable to your body's contours and movements. All-in-all the **SenSura** 1-piece appliance is very discrete both in appearance and design. And the integrated filter also ensures that your stoma appliance will not bother you with ballooning or odors.



A transparent area on the **SenSura** 1-piece allows you to check the stoma condition without removing the appliance. Comes with open bags.

1. Voergaard L, Vendelbo G, Carlsen B et al. Ostomy bag management: comparative study of a new one-piece closed bag. *British Journal of Nursing* 2007; 16(2): 95-101.

# SenSura Flex 2-piece system

Your choice for the unique combination of security and flexibility

Choosing a 2-piece stoma appliance is all about confidence. You have to be confident that the appliance will not let you down by leaking or slipping off. And at the same time it must protect your skin even when left on the skin for days. On top of that an appliance with adhesive coupling must offer you the confidence while allowing you unrestricted freedom of movement. The latest member of the **SenSura** range – **SenSura Flex** – offers you all this and more. The soft and flexible adhesive coupling gives you full confidence and allows you to move freely at the same time. And it makes this 2-piece solution appear as flat and comfortable as a 1-piece appliance.



## An ultra strong adhesive coupling designed for confidence

The unique adhesive coupling keeps the SenSura Flex bag safely and tightly fixed to base plate. This leakage proof adhesion is furthermore ensured by the integrated extra security channels in the coupling that acts as an extra barrier to leakage.

The security channels integrated in the adhesive coupling provides extra protection against leakage of stoma output.



The adhesive Flex coupling and the double layer adhesive of the **SenSura** base plate forms together a very secure solution that fully lives up to your demands for confidence. What is equally important for 2-piece systems is that the high erosion resistance of the **SenSura** adhesive allows you to leave the base plate on your skin for several days with out fearing leakage or skin problems.



## Unrestricted freedom of movement

Thanks to the flat, soft and flexible design **SenSura Flex** offers you total freedom of movement. There are no bulky or stiff elements that get in the way or bother you when you are sitting down and are stretching or twisting your body. And in comparison with other adhesive coupling systems **SenSura Flex** provides you with the unique combination of security and flexibility.

The flat and soft **SenSura Flex** system puts no restrictions on neither physical nor social activities.



**SenSura Flex** system makes it very easy for you to change bags. The used bag is simply peeled off the base plate using the removal ear. And the fresh bag is applied correctly by using the integrated blue guide line.



# SenSura Click

Your choice for reassurance

A 2-stoma appliance with mechanical coupling is the traditional 2-piece solution. This system is preferred by some because of the physical assurance of correct application between the bag and the base plate. **SenSura Click** was developed to fulfill these basic demands to 2-piece stoma appliances by adding a safety-ring to the coupling. This makes **SenSura Click** an excellent choice if you want extra assurance and convenience from your stoma appliance.

## SenSura Click stays safely in place for several days

Thanks to the erosion resistant double layer adhesive the base plate is kept safely and tightly fixed to your skin for several days. And when it is time to change it you simply peel it off without pains and without residues left on your skin.



## With **SenSura Click** you can hear that the coupling is safe

**SenSura Click** is used like a conventional 2-piece system. However, to enhance your personal security the coupling system that connects the bag and the base plate is locked with a special liquid-proof safety ring. The **SenSura Click** system makes it very easy to change bags. And to give you extra reassurance that the connection is right and an audible “click” indicates when the bag is securely fastened and when it can be removed. The great number of people that uses **SenSura Click** today is the best proof of the ability to fulfill the users’ demands for a 2-piece mechanical coupling system.



**SenSura Click** has won several comparative tests thanks to the unique safety ring that tells when the bag is safely locked to the base plate with an audible “click”.<sup>2,3</sup>

## Convenience

**SenSura Click** is perhaps the most convenient mechanical coupling system you can choose. First of all because of the flat coupling that ensures low abdominal pressure and is easy to clean at bag changes. But also because of the bags that are well designed and gentle to the skin.



When the safety ring is open the bag can be turned to another position without removing the bag from the base plate.

2. Arnoldt A, Schröder K, Riedlinger I et al. Management of colostomy with a new 2-piece closed-end appliance. Accepted for publication at the 40th annual Wound Ostomy and Continence Nurses (WOCN) society conference, Orlando, Florida, June 21-25, 2008. Study outcomes on the SenSura 2-piece baseplate adhesive is represented by the clinical benefits demonstrated in the SenSura Click baseplate study.  
3. Schröder K, Fronhoffs-Schommen E, Röthel U et al. Comparative study of a new 2-piece drainable ostomy appliance with convexity. Accepted for publication at the 40th annual Wound Ostomy and Continence Nurses (WOCN) society conference, Orlando, Florida, June 21-25, 2008.